























































Ensure you are applying the right amount of torque for your patient's implant restoration.

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
Astra				
	Astra 3.0	15 Ncm	0.050" (1.27 mm) Hex	
	Astra 3.5-4.0	20 Ncm	0.050" (1.27 mm) Hex	
	Astra 4.5-5.0	25 Ncm	0.050" (1.27 mm) Hex	
	Astra EV 3.0, 3.6, 4.2, 4.8, 5.4	25 Ncm	0.050" (1.27 mm) Hex	
BioHorizons				
	Internal 3.5, 4.5, 5.7	30 Ncm	0.050" (1.27 mm) Hex	
	Tapered Internal 3.5, 4.5, 5.7	30 Ncm	0.050" (1.27 mm) Hex	
	External Implant 3.5	30 Ncm	0.050" (1.27 mm) Hex	
	External Implant 4.0, 5.0	30 Ncm	0.050" (1.27 mm) Hex	
Biomet 3i				
	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	20 Ncm	0.048" (1.22 mm) Hex	
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	20 Ncm	0.048" (1.22 mm) Hex	
	Certain™ XP 4/5, 5/6	20 Ncm	0.048" (1.22 mm) Hex	
	External Hex	35 Ncm	0.048" (1.22mm) Hex	
BlueSkyBio				
	One Stage	30 Ncm	Star & 0.048" Hex	
	Trilobe	30 Ncm	Unigrip & 0.048" Hex	 
	Internal Hex & Molar	30 Ncm	0.050" (1.27mm) Hex	
	Conus 12 & Three	30 Ncm	0.050" (1.27mm) Hex	
	Max	30 Ncm	0.048" (1.22mm) Hex	
	Quatro	30 Ncm	0.048" (1.22mm) Hex	
	OsseoSpeed™ 4.0 - Blue (Previous)	25 Ncm	0.050" Hex	
Dentsply				
	ANKYLOS® C/X - C/3.5, 4.5, 5.5, 7.0 (non-indexed)	15 Ncm	1.0 mm Hex	
	ANKYLOS® C/X - /X 3.5, 4.5, 5.5, 7.0 (indexed)	15 Ncm	1.0 mm Hex	

IMPLANT TORQUE SPECIFICATION GUIDE

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
	Friadent	24 Ncm	0.048" (1.22mm) Hex	
	OsseoSpeed™ EV 3.0 - Green	25 Ncm	0.050" Hex	
	OsseoSpeed™ EV 3.6 - Purple	25 Ncm	0.050" Hex	
	OsseoSpeed™ EV 4.2 - Yellow	25 Ncm	0.050" Hex	
	OsseoSpeed™ EV 4.8 - Blue	25 Ncm	0.050" Hex	
	OsseoSpeed™ EV 5.4 - Brown	25 Ncm	0.050" Hex	
	OsseoSpeed™ Profile EV 4.2 Yellow	25 Ncm	0.050" Hex	
	OsseoSpeed™ Profile EV 4.8 Blue	25 Ncm	0.050" Hex	
	OsseoSpeed™/OsseoSpeed™ TX 3.0 - Yellow	15 Ncm	0.050" Hex	
	OsseoSpeed™/OsseoSpeed™ TX 3.5, 4.0 - Aqua	20 Ncm	0.050" Hex	
	OsseoSpeed™/OsseoSpeed™ TX 4.5, 5.0 - Lilac	25 Ncm	0.050" Hex	
	OsseoSpeed™ 4.0 - Blue (Previous)	25 Ncm	0.050" Hex	
	OsseoSpeed™ TX Profile 4.5, 5.0	25 Ncm	0.050" Hex	
	XIVE® S 3.0	24 Ncm	1.22 mm Hex	
	XIVE® S 3.4, 3.8, 4.5, 5.5	24 Ncm	1.22 mm Hex	
	FRIALIT™ 3.4, 3.8, 4.5, 5.5	24 Ncm	1.22 mm Hex	
Dentium				
	Super Line	30 Ncm	0.050" (1.27mm) Hex	
	Implantium	30 Ncm	0.050" (1.27mm) Hex	
Glidewell				
	iNCLUSIVE® Tapered 3.7, 4.7, 5.2	35 Ncm	0.050" (1.27mm) Hex	
Hiossen/Osstem				
	HG mini 3.5	20 Ncm	0.048" (1.22mm) Hex	
	HG Standard 4.0, 4.5, 5.0	30 Ncm	0.048" (1.22mm) Hex	
Keystone/Lifecore				
	Genesis & Prima	30 Ncm	Square	

IMPLANT TORQUE SPECIFICATION GUIDE

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
	Renova®	30 Ncm	0.048" (1.22mm) Hex	
	Restore®	30 Ncm	0.048" (1.22mm) Hex	
Mega'Gen				
	EZ Plus	35 Ncm	0.048" (1.22mm) Hex	
	Rescue	45 Ncm	0.048" (1.22mm) Hex	
	ExFeel Internal 3.5	25 Ncm	0.048" (1.22mm) Hex	
	ExFeel Internal 4.1, 4.8, 5.5	35 Ncm	0.048" (1.22mm) Hex	
	ExFeel External 3.3	25 Ncm	0.048" (1.22mm) Hex	
	ExFeel External 3.75, 4.0, 4.5, 5.0, 5.5	30 Ncm	0.048" (1.22mm) Hex	
MIS®				
	Biocom & Seven	30 Ncm	0.050" (1.27mm) Hex	
NEODENT®				
	CM	15 Ncm	0.035" (0.9mm) Hex	
	CM	32 Ncm	0.048" (1.22mm) Hex	
	CM	32 Ncm	0.063" (1.6mm) Hex	
Nobel Biocare®				
	Angulated Screw Channel	35 Ncm	Nobel Biocare® Omnigrip	
	NobelActive™ 3.0	15 Ncm	Nobel Biocare® Unigrip	
	NobelActive™/NobelReplace™ Conical Connection NP 3.5	35 Ncm	Nobel Biocare® Unigrip	
	NobelActive™/NobelReplace™ Conical Connection RP 4.3, 5.0	35 Ncm	Nobel Biocare® Unigrip	
	NobelReplace™ NP 3.5	35 Ncm	Nobel Biocare® Unigrip	
	NobelReplace™ RP 4.3	35 Ncm	Nobel Biocare® Unigrip	
	NobelReplace™ WP 5.0	35 Ncm	Nobel Biocare® Unigrip	
	NobelReplace™ 6.0	35 Ncm	Nobel Biocare® Unigrip	
	Brånemark System™ NP 3.3	35 Ncm	Nobel Biocare® Unigrip	
	Brånemark System™ RP 3.75, 4.0	35 Ncm	Nobel Biocare® Unigrip	

IMPLANT TORQUE SPECIFICATION GUIDE

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
	Brånemark System™ WP 5.0, 5.5	35 Ncm	Nobel Biocare® Unigrip	★
Southern Implant				
	External	32 Ncm	0.048" (1.22mm) Hex	⬠
	Tri Nex	32 Ncm	Unigrip	★
	Octa	32 Ncm	Star/Torx	
Straumann®				
	Bone Level 3.3 NC (Narrow CrossFit™)	35 Ncm	SCS Screwdriver	★
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	35 Ncm	SCS Screwdriver	★
	Standard/Standard Plus 4.8 RN (Regular Neck)	35 Ncm	SCS Screwdriver	★
	Standard/Standard Plus 4.8 WN (Wide Neck)	35 Ncm	SCS Screwdriver	★
	Standard Plus 3.3 NN (Narrow Neck)	35 Ncm	SCS Screwdriver	★
Sybron				
	Pitt Easy	25 Ncm	0.067" (1.7mm) Hex	⬠
Thommen				
	SPI® 3.5	15 Ncm	Four Lobe	
	SPI® 4.0, 4.5, 5.0, 6.0	25 Ncm	Four Lobe	
	SPI® Zirconia Abutment	20 Ncm	Four Lobe	
Zimmer Dental				
	Tapered Screw-Vent 3.5, 4.5, 5.7/Tapered Screw-Vent 3.5, 4.5, 5.7 friction fit	30 Ncm	(.050) 1.25 mm Hex	⬠
	Screw-Vent 3.3, 3.7, 4.5/Screw-Vent 3.3, 3.7, 4.5 friction fit	35 Ncm	1.25 mm Hex	⬠
	SwissPlus™ 4.8	35 Ncm	1.25 mm Hex	⬠